



# SURGICAL TIE MASKS

## INSTRUCTIONS FOR USE

### Donning Instructions

1. Remove mask from packaging, ensuring pleats are facing in a downward direction on outer layer facing away from face.
2. Grasp the top and bottom border of the mask and pull to expand pleats.
3. Place mask on face with nosepiece centered on bridge of nose; press nosepiece along cheekbone to mold to face.
4. Adjust mask so that nose and mouth are fully covered.
5. Tie top tie so that it rests at crown of head and bottom tie around neck. Ties should be secure but comfortable.
6. Perform one last press to the nosepiece to ensure it is resting securely against the face.

### Doffing Instructions

1. Release the mask by breaking or untying the bottom tie first.
2. Next, break or untie the top tie.
3. Be sure to hold used mask only by the ties and avoid touching any other part of mask as it is considered contaminated.
4. Dispose of mask in accordance with your facility's guidelines.

**Disclaimer:** *With each don and doff, perform hand hygiene in accordance with your facility's policies.*

Mfd. for  
**S2S GLOBAL**  
Charlotte, NC 28277  
s2s-global.com

  
NOT MADE  
WITH NATURAL  
RUBBER LATEX

  
SINGLE  
USE