

# ReMUV Air Lateral Transfer Device

- Always make sure **Patient is Centered**
  - *Head and heels are on mat or supported*
- Always make sure mat is **Fully Inflated**
  - *Never leave patient unattended when mat is Inflated*
  - *There is min of two caregivers needed for each transfer*
- Always **use with Air Supply**
  - *Leave on for length of transfer and fully deflate prior to pump hose removal*
  - *The pump has a duty cycle on the air supply. 30 secs on, 1 min OFF, 5 cycles with 30 min rest period*
  - *Transfers need to happen soon after full inflation and within 30 seconds to be within the duty cycle.*
- Always **use as Single Patient Use**
  - *Air Lateral Device is designed to use used multiple times on same patient during their hospital stay*
  - *Once device is unzipped the mat is no longer usable and is to be discarded*

