

ReMUV Air Lateral Transfer Device

Always make sure **Patient is Centered**

- *Head and heels are on mat or supported*

Always make sure mat is **Fully Inflated**

- *Never leave patient unattended when mat is inflated*
- *There is min of two caregivers needed for each transfer*

Always **use with Air Supply**

- *Leave on for length of transfer and fully deflate prior to pump hose removal*
- *The pump has a duty cycle on the air supply. 30 secs on, 1 min OFF, 5 cycles with 30 min rest period*
- *Transfers need to happen soon after full inflation and within 30 seconds to be within the duty cycle.*

Always **use as Single Patient Use**

- *Air Lateral Device is designed to use used multiple times on same patient during their hospital stay*
- *Once device is unzipped the mat is no longer usable and is to be discarded*



HONORHEALTH

HONORHEALTH

Air Supply Operation Instructions

1. Ensure the power switch is in the off (o) position
2. Plug the air supply into a 120AC power outlet
3. Connect the air supply hose to the REMUV™ Air Mat
4. Turn the Power Switch at the top of the Air Supply to the on (I) position (one power output with continuous flow)
5. Turn the Power Switch at the top of the Air Supply to off (o) position after use
6. Disconnect the air supply hose from REMUV™ Air Mat
7. Unplug the air supply power cord from wall outlet and store.



Password:

Honor

NOTE: DUTY CYCLE: 30 sec ON, 1 min OFF, 5 cycles with 30 min rest period

Air Supply Operation Instructions

1. Ensure the power switch is in the off (o) position
2. Plug the air supply into a 120AC power outlet
3. Connect the air supply hose to the REMUV™ Air Mat
4. Turn the Power Switch at the top of the Air Supply to the on (I) position (one power output with continuous flow)
5. Turn the Power Switch at the top of the Air Supply to off (o) position after use
6. Disconnect the air supply hose from REMUV™ Air Mat
7. Unplug the air supply power cord from wall outlet and store.



Password:

Honor

NOTE: DUTY CYCLE: 30 sec ON, 1 min OFF, 5 cycles with 30 min rest period

ReMUV Air Lateral Transfer Device

Always make sure **Patient is Centered**

- *Head and heels are on mat or supported*

Always make sure mat is **Fully Inflated**

- *Never leave patient unattended when mat is inflated*
- *There is min of two caregivers needed for each transfer*

Always **use with Air Supply**

- *Leave on for length of transfer and fully deflate prior to pump hose removal*
- *The pump has a duty cycle on the air supply. 30 secs on, 1 min OFF, 5 cycles with 30 min rest period*
- *Transfers need to happen soon after full inflation and within 30 seconds to be within the duty cycle.*

Always **use as Single Patient Use**

- *Air Lateral Device is designed to use used multiple times on same patient during their hospital stay*
- *Once device is unzipped the mat is no longer usable and is to be discarded*

